



FreeWheel® Folding Wheelchair Adaptor: Set-Up Instructions

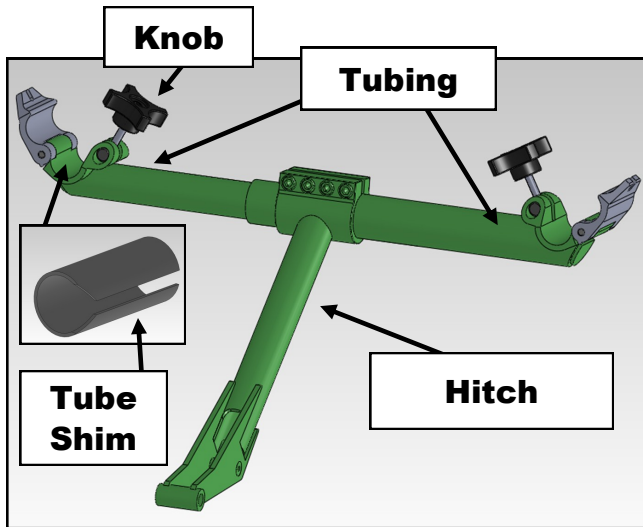

NOTE: Video instructions under VIDEOS at www.gofreewheel.com

STEP 1: Getting Started


- A. Separate tubing
- B. Slide HITCH on to one side of tubing. Slide tubing back together.
- C. Open Knobs and remove Tube Shims.

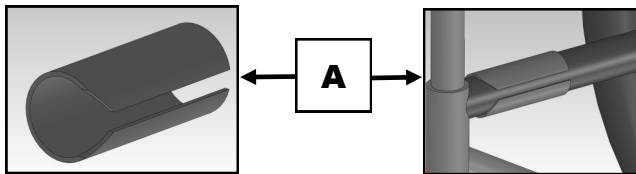


STEP 2: Securing Adaptor to Your Wheelchair

- A. Clip Tube Shims onto horizontal tubing.


Note: Tube Shims are not required on 1-18" tubing.

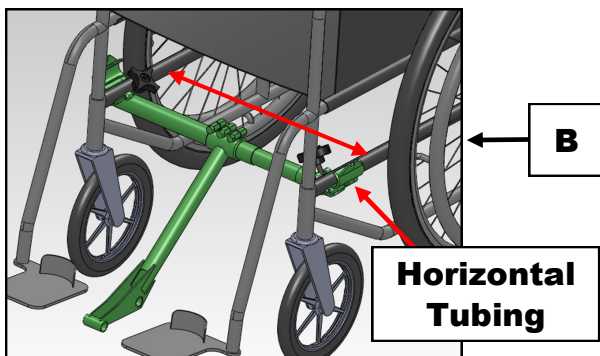


- B. Slide Adaptor tubing to width of wheelchair and close clamps over wheelchair horizontal tubing.

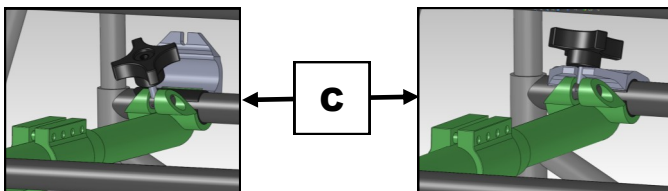

- C. Secure clamps by lifting knob into slot and tightening.

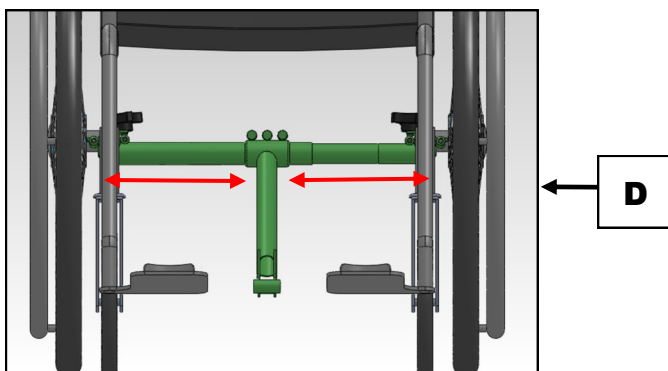


- D. Move hitch so it is perfectly centered between horizontal tubing.

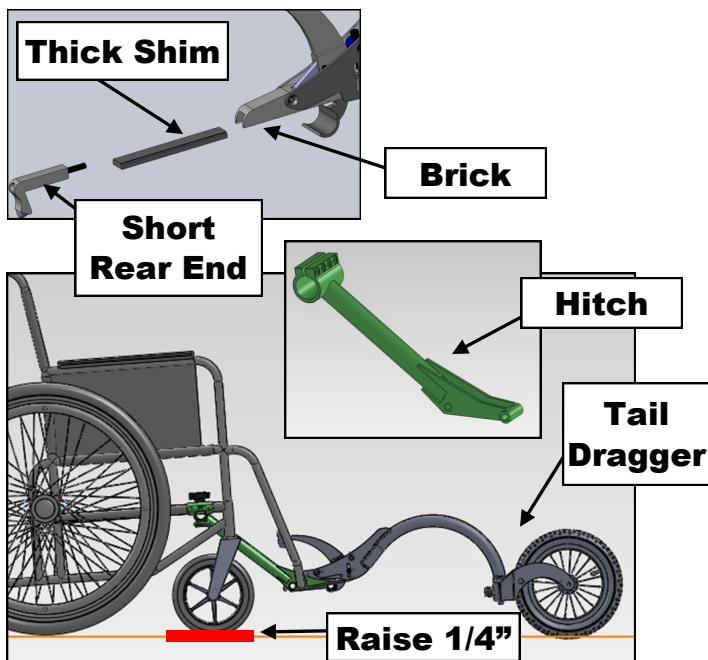


STEP 3: Attaching FreeWheel

- Cut 1" off thickest straight shim from FreeWheel kit and slide into FreeWheel brick.
- Insert shortest Rear End from FreeWheel kit and screw into brick. Do not tighten completely but leave small gap.
- Raise wheelchair front casters approximately 1/4".
- Clamp FreeWheel on to Hitch with FreeWheel in Tail Dragger position.

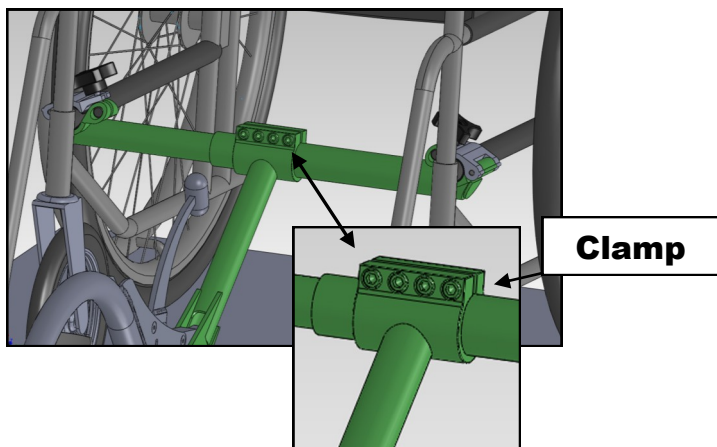

STEP 4: Final Adjustments

- Start by SLIGHTLY tightening all 4 of the screws on the clamp. It is important to not completely tighten each screw, but to slowly tighten all 4, a little at a time.
- Keep tightening until all four screws are very secure. (20 ft-lbs torque).
- Sit in chair and rotate tire under the FreeWheel frame.
- If your toes hit the wheel, loosen clamps and move Adaptor forward on the horizontal bars.

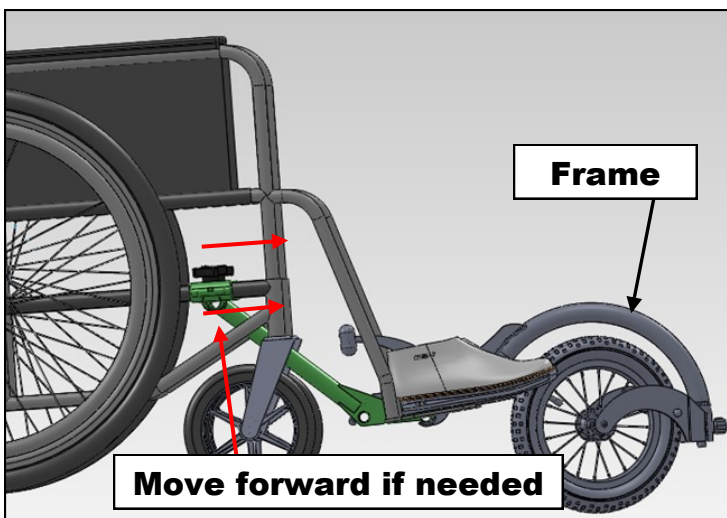

Please refer to the FreeWheel Set-up Guide to fine tune any adjustments.

For more information or questions, contact us at:

Email: info@gofreewheel.com Web: www.gofreewheel.com